

The original bathing and skin cleansing breakthrough

BagBath®



Instructions for Warming BagBaths in a Microwave



Place the unopened BagBath in the microwave and heat for 15-30 seconds, keeping in mind that microwaves heat at varying rates. Use caution not to over-heat the BagBath.

Instructions for Use of Warming Units



30-Unit Warmer

- 1 Once unpacked, the warming unit should be placed upright with the door opening toward the user. Connect the cord into a 110/120 Volt AC wall outlet. Warming unit may be run continuously.
- 2 Fill warming unit with the appropriate number of unopened BagBath packages. They will be warm and ready to use at optimum temperature in approximately 4 - 6 hours.

Model 10012W holds 12 BagBaths / Model 10020W holds 30 BagBaths
- 3 Refill the warming unit at least 5 hours prior to bath-time to ensure an adequate supply of BagBath packages.
- 4 To ensure stock rotation, it is recommended that the user selects a BagBath package from the bottom of the stack in the warming unit. To restock, place a new BagBath package on top of the existing warmed supply.

Additional Information: Warming Unit

Power Source: AC 115V (+/-10%) 60 HZ, 1.1 amps. (10012W - 12 BagBath Warmer)/2.2 amps (10030W - 30 BagBath Warmer);
Maximum Temperature: 52° C/125° F; **Compliance:** UL Standard 3101 1 IEC 61010-1. Underwriter's Laboratory Listed, Laboratory Equipment, UL/CUL United States/Canadian, File No. E212550

Contact

407-405-7722 888-838-9163 www.BagBath.com info@BagBath.com

Health Care • Nursing Facilities • Home Care • Athletics • Outdoor Enthusiasts